

# SuperDART

## Endoscope Channel Cleaning Device

Novel three stage single use  
endoscope channel cleaning device

Channels cleaned in just ONE PASS™  
Validated cleaning efficacy for ONE PASS™  
Easy pull through action


### Key Characteristics

Pull through three stage  
SuperDART® is the most  
sophisticated endoscope  
channel cleaning device on  
the market today


**1st stage - Robust leading brush:**  
dislodges the debris off the  
channel wall

**2nd stage - Multi-disk squeegee  
wiper:** pushes the debris out and  
swipes the channel wall

**3rd stage - Fine bristled tail brush:**  
dislodges the debris from the channel  
wall imperfections and brushes off  
the remaining biofilm before the final  
flush with a detergent solution

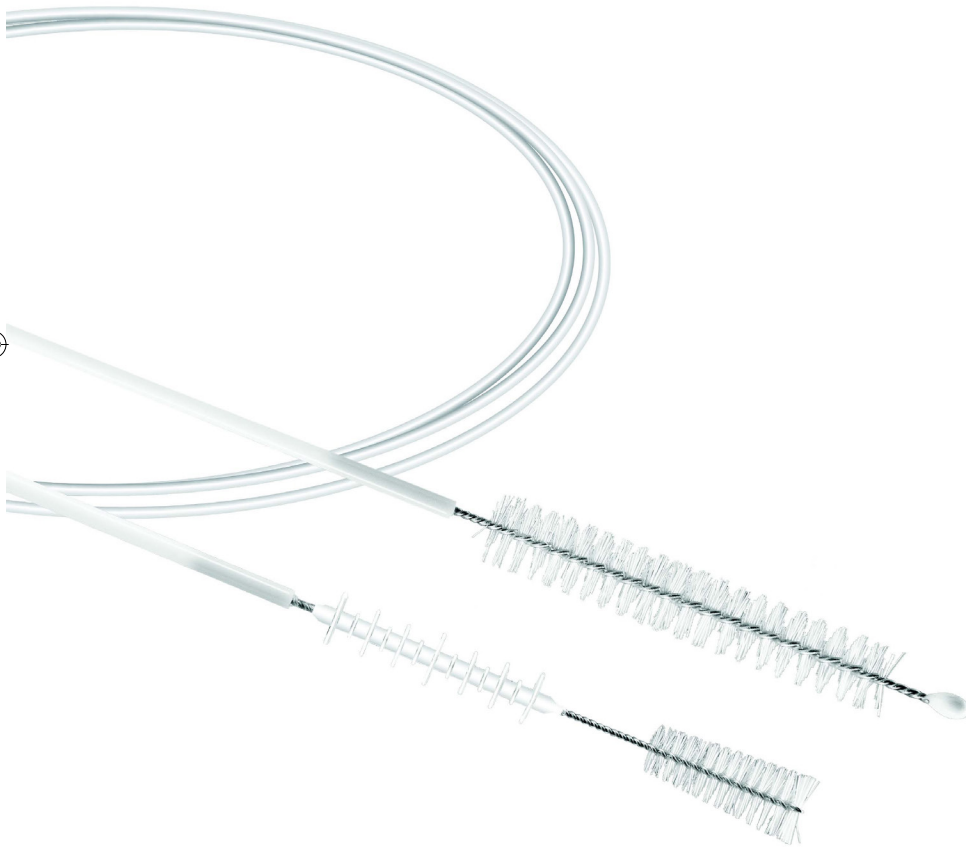

This is how SuperDART works:

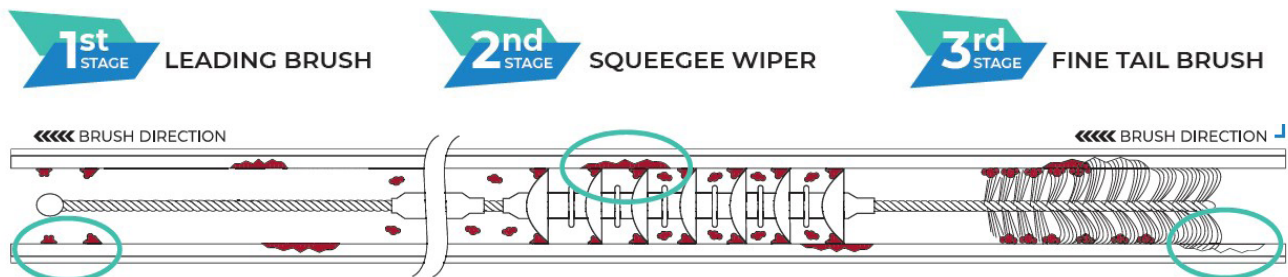

**SuperDART® ONE PASS™** achieves validated **99.997 - 100% soil removal** from contaminated channels in a single swipe<sup>1</sup>

SuperDART® ONE PASS™ Advantage	Standard Brush
Time savings with a single swipe	Multiple passes required
Operators' fatigue reduction	Repetitive strenuous work
Objective high level of cleanliness	Subjective and uncertain cleanliness

(1) Declaration of Performance and raw validation data available upon request.

SuperDart - Endoscope Channel Cleaning Device							
Reference	Description	Sheath Diameter (mm)	Length (cm)	Brushes Diameter (mm)	Squeegee Wiper Diameter (mm)	Channel Size (mm)	Box QTY
9292	SuperDART® 3 Cleaning Stages Tool	1.7	250	Leading Brush: 5-7 Tail Brush: 4 - 6.3	2.5 - 5.2	2.8 - 4.2	25
9188	SuperDART® 3 Cleaning Stages Tool with Valve/ Control Head Brush	1.7	250	Leading Brush: 5-7 Tail Brush: 4 - 6.3 Valve/Control Head Brush: 10	2.5 - 5.2 (Squeegee Wiper)	2.8 - 4.2	25

**For ordering information in the UK  
please contact Customer Service:**

**Creo Medical Ltd.**  
Unit 2, Creo House, Beaufort Park Way  
Chepstow NP16 5UH, UK  
+44 (0) 1291 637 300  
sales@creomedical.com

**Product  
Ordering**

[in showcase/creo-endoscopy](#) [in company/creo-medical](#) [X @CreoMedical](#) [X @CreoEndoscopy](#)

[creomedical.com](http://creomedical.com)

Technical changes and mistakes reserved. Images may show non standard options.

Creo Medical, the CM logo, Kamaptive and Speedboat are trademarks of Creo Medical Limited. Certain names and marks are registered in the UK and other territories. All cited trademarks are the property of their respective owners. © 2024 Creo Medical. All rights reserved.

MKT-OEM-GIH-001 | 2024-July

